



Techno-Aide, Inc.
7117 Centennial Boulevard
Nashville, TN 37209
P: 800.251.2629
F: 800.554.6275

Leg Holder CLEANING & CARE INSTRUCTIONS 2016

Thank you for choosing Techno-Aide for your radiology accessory needs. To ensure the longevity of your new Leg Holder, we recommend the following guidelines for cleaning and care.

Cleaning:

We all know that in a medical environment, it is easy for positioning aides to become dirty and soiled from blood, bodily fluids, betadine, barium, and other fluids.

Do:

- Techno-Aide recommends cleaning leg pad holder with cold water and a mild detergent to remove all immediate contacts with fluids.
- 409 Spray can be used to clean the surface fabric and remove dust and dirt from the leg pad holder.
- To disinfect your leg holder and base, Techno-Aide recommends the use of disinfectant wipes such as PDI Sani-Cloth Germicidal Wipes Cat# GWP-WAF. Allow a drying time of at least 3 minutes.

Don't:

- Do not use Bleach or any harsh chemicals on the leg pad holder. The base can be wiped with a Bleach product.
- Do not autoclave leg pad holder
- Do not soak/submerge leg pad holder in water.
- Do not launder leg pad holder.

Failure to adhere to the above recommendations will shorten the life of your leg holder. Techno-Aide provides a one year warranty on workmanship and materials for our radiology accessory products.



Techno-Aide, Inc.
7117 Centennial Boulevard
Nashville, TN 37209
P: 800.251.2629
F: 800.554.6275

CARE:

Radiation protective apparel and shields are sometimes inadvertently damaged when not properly handled or stored. The following guidelines may be used to help achieve this goal:

Do:

- Before use, inspect for defects, tears, perforations, creases.

Don't:

- Don't exposure your shield to extreme temperatures of hot or cold or to direct sunlight.
- Do not drape items over shields.
- Do not lean against sharp edges or objects with sharp points.

Failure to adhere to the above recommendations will shorten the life of your shield, promote cracking, and will void your warranty. Techno-Aide provides a two year warranty on workmanship and materials for our protective apparel garments

DISPOSAL:

Under the Resource Conservation and Recovery Act (RCRA) and the Environmental Protection Agency (EPA) Lead is included in the Toxic substance list. It should not be disposed of as conventional garbage but as hazardous waste. Coordinate with the radiation control officer in your organization who is responsible for disposal of such wastes. Techno-Aide recycles only Techno-Aide manufactured apparel and shields. If you currently have a TA product that needs to be disposed of, simply ship it back to us and we will dispose of it properly at no cost to you (other than shipping).

If you have any Questions regarding your Techno-Aide Radiation Protection Shield Purchase, please contact your Local X-Ray Dealer or Techno-Aide Customer Service.