



"We Do Care - We Do More"

MJM International Corporation

323 East Owassa • Suite 4 • Edinburg, TX 78539 • P.O. Box 720054 • McAllen, Texas 78504
1-800-270-6199 • 956-781-5000 • Fax: 956-781-5552 • www.mjminternational.com • sales@mjminternational.com

MJM CARE INSTRUCTIONS

PVC PIPING AND CASTERS

Wipe down piping with non-abrasive cleanser. **DO NOT** use solvents. Restore luster with Johnson's Pledge or similar spray wax. If casters are used in wet environment, keep lubricated with white lithium paste or similar lubricant; remove hair and mop- thread build up. Remove casters at least once a week to avoid premature rusting

MESH FABRIC (SLING)

If laundering is required, use only **COLD** water in large machine, with regular detergent and one-cup bleach. Do not use softeners or solvents. Spin-dry only; machine heat will shrink mesh fabric. Hang damp fabric on frame and allow to air dry, returning to its original shape.

SOLID FABRIC (PRIVACY SKIRTS)

Wipe down panels with sponge using solution of mild detergent, one-cup bleach and three gallons water, **DO NOT** launder in machine. To keep vinyl flexible, without drying out and cracking, occasionally spray with furniture polish (e.g. Pledge) or vinyl formula (Armor All).

FOOTREST AND DROP ARMS

Remove soap build-up from sliding surface; with a soft bristle brush and soapy solution. Lubricate sliding mechanism with furniture spray wax to restore ease in movement.

When lifting drop-arm, use two hands, one at each end of the section. Lifting only at hand grip will put undue pressure against the sliding fitting.

SOFT SEAT or PADDED BACK (USING SAME SOFT SEAT MATERIAL) MARINE VINYL

In general, most common stains can be easily cleaned using warm, soapy water and clear water rinses. Moderate scrubbing with soft bristle brush will help to loosen soiling materials from the depressions of embossed surfaces.

CAUTION: **DO NOT** use chair if casters are rusted and fail to move easily or if cracks occur in fittings.

"WE DO CARE- WE DO MORE"