Super Sani-Cloth®

GERMICIDAL DISPOSABLE WIPE
LINGETTE GERMICIDE JETABLE

LARGE WIPE
GRANDE LINGETTE

BACTERICIDAL, TUBERCULOCIDAL, AND VIRUCIDAL * IN 2 MINUTES
BACTÉRICIDE, TUBERCULOCLIDE, VIRUCIDE * EN 2 MINUTES

ACTIVE INGREDIENTS:

n-Alkyl (68% C₁₂, 32% C₁₄) dimethyl ethylbenzyl ammonium chlorides.....0.25%

n-Alkyl (60% C₁₄, 30% C₁₆, 5% C₁₂, 5% C₁₈) dimethyl benzyl ammonium chlorides.....0.25%

Isopropyl Alcohol.....55.00%

OTHER INGREDIENTS.....44.50%

TOTAL.....100.00%

Does not include the weight of the wipe.

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chlorures de n-alkyl (68% C₁₂, 32% C₁₄) alkyldiméthyl (éthylbenzyl) ammonium.....0,25%

chlorures de n-alkyl (60% C₁₄, 30% C₁₆, 5% C₁₂, 5% C₁₈) alkyldiméthyl (éthylbenzyl) ammonium.....0,25%

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6 x 6.75 in (15.2 x 17.1 cm) / 15.2 x 17.1 cm (6 x 6.75 po)

Net Wt. 1 lb 13 oz (833g) / Poids net 1 lb 13 oz (833 g)

Effective against / Efficace contre :

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CHAMPIGNONS PATHOGÈNES • *Candida albicans*

8M055100

*KILLS HIV-1 (AIDS VIRUS), HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) ON THE PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS AND PRE-CLEANED SURFACES OF ULTRASOUND TRANSDUCERS AND PROBES PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS IN TWO (2) MINUTES AT ROOM TEMPERATURE (68°-77°F) IN HEALTHCARE OR OTHER SETTINGS IN WHICH THERE IS AN EXPECTED LIKELIHOOD OF SOILING OF INMATE SURFACES/OBJECTS WITH BLOOD OR BODY FLUIDS; AND IN WHICH THE SURFACES/OBJECTS ARE LIKELY TO BE SOILED WITH BLOOD OR BODY FLUIDS CAN BE ASSOCIATED WITH THE POTENTIAL FOR TRANSMISSION OF HUMAN IMMUNODEFICIENCY VIRUS TYPE 1 (HIV-1) (ASSOCIATED WITH AIDS), HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV).

Areas of Use: Hospital and Healthcare Settings: Ambulatory Surgical Centers (ASC), Clinics, Dental Offices, Dialysis Clinics, Home Health Care, Hospices, Hospitals, Laboratory, Nursing Homes, Physical Therapy, Physicians' offices, Radiology, Rehabilitation, Transport vehicles. Critical Care Areas: CCU, Emergency Rooms, ICU, Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU), Surgery and Surgical Intensive Care Unit (SICU), Hospital, Healthcare, and Critical Care Use Sites. May be used on hard non-porous surfaces of: bed railings; blood glucose meters; cabinets; carts; chairs; counters; dental unit instrument trays; exam tables; gurneys; isolettes; IV poles; stethoscopes; stretchers; tables; telephones; toilet seats; and hard non-porous outside surfaces of: amalgamators and dental curing lights; diagnostic equipment; patient monitoring equipment; patient support and delivery equipment. This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection. DIRECTIONS FOR USE: It is a violation of Federal law to use this product in a manner inconsistent with its labeling. To Dispense Wipes: Remove lid and discard inner seal from canister. Find center of wipe roll, remove first wipe for use, twist corner of next wipe into a point and thread through the hole in the canister lid. Pull through about one inch. Replace lid. Dispense remaining wipes as necessary by pulling out at an angle. When not in use keep center cap of lid closed to prevent moisture loss. TO DISINFECT AND DEODORIZE: To disinfect nonfood contact surfaces only: Unfold a clean wipe and thoroughly wet surface. Allow treated surface to remain wet for two (2) minutes. Let air dry. For heavily soiled surfaces, use a wipe to pre-clean prior to disinfecting. These directions also apply to *Mycobacterium bovis* BCG (Tuberculosis) at 68° F (20° C). *SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS. Personal protection: When using this product, wear disposable protective gloves, protective gowns, masks, and eye coverings when handling HIV-1 (AIDS Virus), HBV or HCV infected blood or body fluids. Cleaning procedure: All blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfection by the germicidal wipe. Open, unfold and use first germicidal wipe to remove heavy soil. Disposal of infectious materials: Used wipe, blood and other body fluids should be disposed of according to local regulation for infectious waste disposal. Contact time: Use second germicidal wipe to thoroughly wet surface. Allow surface to remain wet two (2) minutes; let air dry. PRECAUTIONARY STATEMENTS: Hazards to Humans and Domestic Animals. WARNING. Causes substantial but temporary eye damage. Do not get in eyes or on clothing. Avoid contact with skin. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using restroom. Remove and wash contaminated clothing before reuse. FIRST AID Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. If on skin: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. PHYSICAL OR CHEMICAL HAZARD Combustible. Do not use or store near heat or open flame. Do not use on natural marble, windows, unpainted wood, brass, clear plastic or colored glass. Test wipe on small inconspicuous area first. STORAGE AND DISPOSAL Do not contaminate water, food, or feed by storage and disposal. Storage: Do not store near heat or open flame. When not in use keep center cap of lid closed to prevent moisture loss. Towellette Disposal: Do not reuse towellette. Dispose of used towellette in trash. Do not flush in toilet. Container Disposal: Nonrefillable container. Do not reuse or refill this container. Offer for recycling. If recycling is not available, put in trash collection. *TUE LE VIH-1 (VIRUS DU SIDA), LE VIRUS DE L'HEPATITE B (VHB) ET LE VIRUS DE L'HEPATITE C (VHC) SUR LES SURFACES/OBJECTS ENVIRONNEMENTAUX PRÉ-Nettoyés ET LES SURFACES EXTERNES PRÉ-Nettoyées DES TRANSDUCTEURS ULTRASONS ET DES SONDES PRÉCEDEMMENT SOUILLEES DE SANG/DE FLUIDES CORPORELS en deux (2) minutes à température ambiante

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